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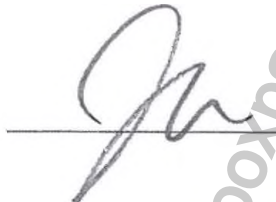
19.12.2024
«Approved»

(signature)
Kim Beom Jun,
CEO
Dental Studio Co. Ltd.

**INSTRUCTIONS FOR USE
FOR THE MEDICAL DEVICE**

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Karl Fritz® scalpel blade handle

I. Design options:

1. Handle No. 3 for scalpel blades, flat, P/N 4601 – 1 pc/pack;
2. Handle No. 5 for scalpel blades, round, straight, P/N 4602 – 1 pc/pack;
3. Handle No. 5A for scalpel blades, round, curved, P/N 4603 – 1 pc/pack;
4. Handle No. 3 for scalpel blades, round, rotary, P/N 4604 – 1 pc/pack;
5. Handle No. 3 Black Double for scalpel blades, round, straight, P/N 4605B – 1 pc/pack;
6. Handle No. 3 Black Double for scalpel blades, round, straight, double-sided, P/N 4606 – 1 pc/pack;
7. Handle No.3 EASY scalpel blade holder, round straight, P/N 4607-1 – 5 pc/pack

II. Instructions for use – 1 pc./pack. (in paper and electronic form)

Hereinafter — instruments, devices, medical device.

2. Information about the manufacturer, developer and authorized representative of the manufacturer

Manufacturer / Developer:

Dental Studio Co., Ltd.

B101, B102, B125, B1, Geumjeong Station 2nd SKV1 Tower, 136 LS-ro, Dongan-gu, Anyang-si, Gyeonggi-do, Republic of Korea

Tel.: +82-31-8069-7007

Fax: +82-31-8069-7009

E-mail: bjkim@dentalstudio.co.kr

Web-site: www.dentalstudio.co.kr

Manufacturer's authorized representative in the Russian Federation

DENTALSTOM LLC

143432, Moscow region, Krasnogorsk, Nakhbino work settlement, Chkalova Str. 7, space LXXV

Tel./fax: +7 (495) 118-76-40

E-mail: info@dn-stom.ru

Web-site: www.dn-stom.ru

3. Intended use of the device/Indications.

Used as a handle into which a blade compatible with that handle is installed.

4. Potential consumers.

Dental surgeons, maxillofacial surgeons and implantology surgeons.

5. Scope of application, contraindications and possible side effects

Scope of application: dentistry, maxillofacial surgery, dental surgery.

Contraindications: possible allergic reaction to materials.

Side effects: Not known.

6. Precautions.

1. The instruments are supplied in a non-sterile package. It is necessary to sterilize the instruments before using them.

2. Check the instruments for mechanical damage. Before using each instrument, the specialist must make sure that the instruments are in working condition; their surfaces shall be free of any contaminations.

3. The instruments must be used in accordance with their intended purpose and instructions for use.

4. This device can be used only by dentists who received qualifications in the application of appropriate treatment methods and completed training courses.

5. Surgical interventions should be carried out in sterile conditions, medical staff should use masks, gloves, sterile clothes.

7. Description, design of devices and application procedure.

Used as a handle into which a blade compatible with that handle is installed. These are reusable products.

Description: Handles for scalpel blades are a component of the scalpel. The handles are intended to hold scalpel blades on them for cutting or excision of soft tissue.

Product name	Description	Appearance
Handle No. 3 for scalpel blades, flat, P/N 4601;	Handle shape is flat. Applied part is straight, designed for installation of one blade.	
Handle No. 5 for scalpel blades, round, straight, P/N 4602;	Handle shape is round. Applied part is straight, designed for installation of one blade.	
Handle No. 5A for scalpel blades, round, curved, P/N 4603;	Handle shape is round. Applied part is curved, designed for installation of one blade.	
Handle No. 3 for scalpel blades, round, rotary, P/N 4604;	Handle shape is round. Applied part is rotary, designed for installation of one blade.	

Product name	Description	Appearance
Handle No. 3 Black Double for scalpel blades, round, straight, P/N 4605B;	Handle shape is round. Applied part is straight, designed for installation of one blade.	
Handle No. 3 Black Double for scalpel blades, round, straight, double-sided, P/N 4606.	Handle shape is round. Applied parts are straight, designed for installation of two blades at the same time.	
Handle No.3 EASY scalpel blade holder, round straight, P/N 4607-1	Handles for scalpel blades are an integral part of a scalpel. The handles are designed to fix scalpel blades on them in order to cut or excise soft tissues.	

Technical specifications

Product name	Total length, mm	Connection characteristic	Material
Handle No. 3 for scalpel blades, flat, P/N 4601;	125.0	Small	Stainless steel
Handle No. 5 for scalpel blades, round, straight, P/N 4602;	154.0	Small	Stainless steel
Handle No. 5A for scalpel blades, round, curved, P/N 4603;	150.0	Small	Stainless steel
Handle No. 3 for scalpel blades, round, rotary, P/N 4604;	160.0	Small	Stainless steel
Handle No. 3 Black Double for scalpel blades, round, straight, P/N 4605B;	160.0	Small	Stainless steel
Handle No. 3 Black Double for scalpel blades, round, straight, double-sided, P/N 4606.	140.0	Small	Stainless steel
Handle No.3 EASY scalpel blade holder, round straight, P/N 4607-1	143.0	Small	Stainless steel

The procedure for using handles for scalpel blades:

1. Clean, disinfect and sterilize the device in accordance with the paragraph 11 of this document;
2. Select an instrument that is suitable in size and purpose;
3. Attach the scalpel blade to the handle and cut or excise soft tissue, holding the instrument by the handle.

The procedure for using EASY scalpel blade holders, P/N 4607

1. Clean, disinfect and sterilize the device in accordance with the paragraph 11 of this document.
2. Loosen the crimp nut on the handle.
3. Pull out the spring-loaded applied part and fix the scalpel blade on it.
4. Tighten the crimp nut on the handle so that the applied part with the scalpel blade is stationary.
5. At the end of working with the instrument, loosen the crimp nut on the handle, pull out the spring-loaded part, remove the blade from the applied part

8. Package contents

Karl Fritz® scalpel blade handle

- I. Design options:
 1. Handle No. 3 for scalpel blades, flat, P/N 4601 – 1 pc/pack;
 2. Handle No. 5 for scalpel blades, round, straight, P/N 4602 – 1 pc/pack;
 3. Handle No. 5A for scalpel blades, round, curved, P/N 4603 – 1 pc/pack;
 4. Handle No. 3 for scalpel blades, round, rotary, P/N 4604 – 1 pc/pack;
 5. Handle No. 3 Black Double for scalpel blades, round, straight, P/N 4605B – 1 pc/pack;
 6. Handle No. 3 Black Double for scalpel blades, round, straight, double-sided, P/N 4606 – 1 pc/pack;
 7. Handle No.3 EASY scalpel blade holder, round straight, P/N 4607-1 – 5 pc/pack
- II. Instructions for use – 1 pc./pack. (in paper and electronic form)

9. Decoding of symbols used in packaging labeling

Symbol	Meaning	Symbol	Meaning
	Catalogue number		Caution
	Lot code		Read instructions for use or see electronic instructions for use
	Date of manufacture		QR-code
	Use before		Non-sterile
	Manufacturer		

10. Conditions of storage, transportation and operation

Storage conditions:

- store in a dry place protected from direct sunlight;
- temperature: +1 to +40 °C;
- relative humidity: 45-55 %;
- atmospheric pressure: 800-1 060 hPa.

Transportation conditions:

Transportation of devices packaged into a transport container shall be carried out by all types of covered vehicles in compliance with the precautions indicated on the transport container and in accordance with the rules of cargo transportation applicable to that type of transport.

- temperature: -50 to +50 °C;
- relative humidity: 0–90 %;
- atmospheric pressure: 800-1 060 hPa.

Indoor conditions during medical device use

- temperature: +1 to +40 °C
- relative humidity: 45-55 %;
- atmospheric pressure: 800-1 060 hPa.

11. Cleaning, disinfection and sterilization of devices

Used instruments shall be cleaned, disinfected and sterilized before being sent for storage.

1. Pre-sterilization cleaning

Instruments shall be cleaned and disinfected before sterilization.

- 1) Immerse the instruments with the remaining impurities (blood, soft tissues, bone fragments, etc.) in water. The instruments shall be kept in water before the following cleaning steps are started manually or in an ultrasonic bath.

2-1) Manual cleaning and disinfection of the instruments

a. Immerse the instruments with the remaining impurities in an enzyme disinfectant detergent diluted in water (it is recommended to use products with a neutral pH index or a disinfectant cleaner with similar characteristics (all products must be registered in accordance with the established procedure in the Russian Federation)).

b. Cleaning of all visible contaminations from the surface of surgical instruments is performed using a soft nylon brush with a short pile (cleaning time of each instrument is at least 4 minutes). Cleaning of the delicate applied parts of the instruments is carried out using cleaning needles.

c. The instruments are rinsed under running water to remove any remaining disinfectant.

2-2) Cleaning and disinfection of instruments in an ultrasonic bath (recommended by the manufacturer as the preferred method)

a. The instruments shall be completely immersed in the cleaning solution. It is recommended to use an enzyme disinfectant cleaner (it is recommended to use products with a neutral pH index or a disinfectant cleaner with similar characteristics (all products must be registered in accordance with the established procedure in the Russian Federation)).

b. The instruments are to be cleaned in an ultrasonic bath for 10–20 minutes, then rinsed with water.

c. The instruments are to be rinsed under running water to remove any remaining cleaning disinfectant.

- 3) Repeat steps a) and b) until the instruments are completely cleaned, then perform visual inspection of the instruments.

4) The instruments are thoroughly dried naturally or using drying cabinets for surgical instruments. If necessary, manual drying using napkins made of non-abrasive lint-free fabric is possible.

5) For more information and data on using the available enzyme and non-enzyme disinfectants, refer to the instructions of the manufacturers of these products.

2. Steam sterilization

After cleaning and disinfecting the instruments, it is necessary to carry out sterilization.

The instruments shall be sterilized by steam under pressure in an autoclave at a temperature from 132 °C to 134 °C for 15 minutes, followed by drying for 30 minutes.

12. Disposal

Medical devices shall be disposed of in accordance with the medical procedures adopted by the medical institutions, and the local, state, or federal regulations. The choice of methods for the safe disposal and destruction of devices depends on the capacity and profile of the medical organization, the availability of waste disinfection/neutralization facilities, the method of neutralization/destruction of waste adopted in the administrative territory (incineration, removal to landfills, disposal).

Instruments are considered epidemiologically hazardous waste (devices contaminated with blood and/or other biological fluids of patients, as well as devices in contact with patients with infectious diseases). In this case, a medical organization only collects and temporarily stores the above-specified waste. Collection is carried out into single-use bags or containers with color labeling adopted for hazardous medical waste in a room or in places for temporary storage of medical waste before subsequent transportation to the place of destruction or disposal. Instruments in contact with the patient shall be pre-cleaned and disinfected in accordance with the established procedure before being disposed of. Transportation of medical waste from medical institutions, and their disposal is carried out by specialized organizations having a license to handle epidemiologically hazardous medical waste.

Consumer package and unused defective devices belong to epidemiologically safe waste, similar in composition to solid household waste (SHW).

13. Shelf life

The shelf life of the device is unlimited.

14. Service life of devices

With proper use, the service life of the instruments is unlimited, except for the effects of natural wear.

15. Maintenance and repair

Maintenance and repairs are not provided.

16. List of international standards that the medical device complies with

EN ISO 13485; EN ISO 14971; EN ISO 14155-1; EN ISO 14155; EN ISO 10993-5; EN ISO 15223-1; EN 1041; MEDDEV 2.7/1 Rev. 4; MEDDEV 2.12/1 Rev. 8; EN ISO 17665-1; EN ISO 11138-1; EN ISO 11737-1; ISO 17665-1; ISO 11135-1; ISO 13397-2.

Note: The latest editions of the standards are used; the organization maintains and updates the list of reference standards as a record of its own management system in accordance with ISO 13485.

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LEE, JUNGYONG -----

attorney-in-fact of
DENTAL STUDIO CO., LTD.

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appeared before me and admitted
said principal's subscription to
the attached

INSTRUCTIONS FOR USE FOR
THE MEDICAL DEVICE. -----

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This is hereby attested on this
20th day of Jan. 2025 at this office.

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This office has been authorized by the
Minister of Justice, the Republic of
Korea, to act as Notary Public Since
7, Feb. 2020 Under Law No.15150.

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